
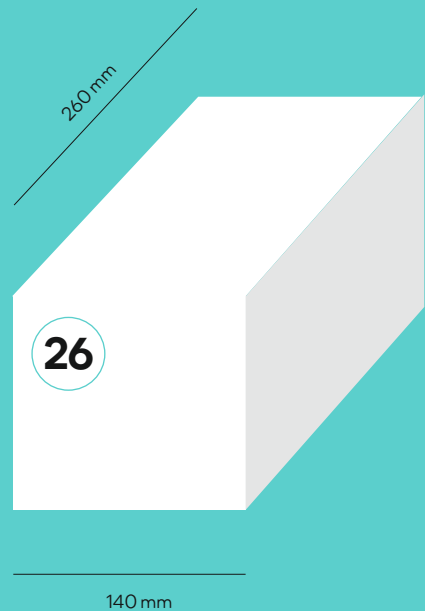
36/26 Manual

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The Anelabel dispenser serves for holding and unwinding of rolls with Anelabel labels in medical facilities.

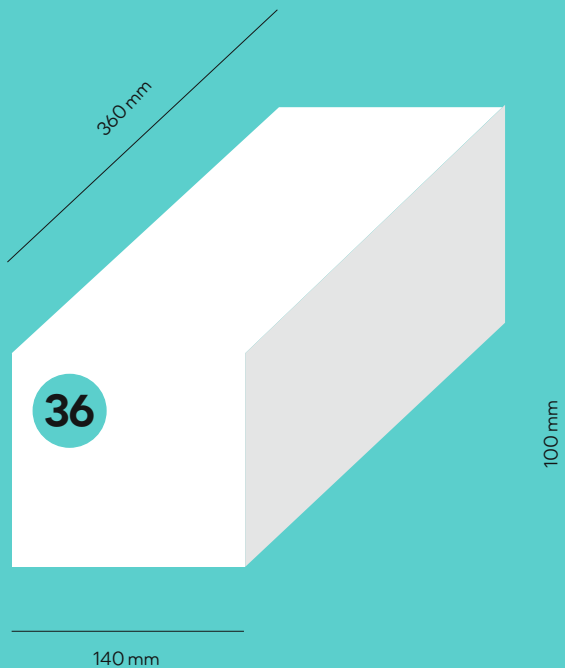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EN

Unpacking the dispenser

Once you remove the dispenser from the package, stick the “Do not clean with alcohol” sticker on a clearly visible place on the dispenser. Then it is recommended to clean the Anelabel dispenser with an **ALCOHOL-FREE** cleaning agent.

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EN

Loading label rolls

Draw out the drawer with partitions. Choose the number of partitions in the drawer to correspond to the selected number and widths of label rolls. Place the first roll in the drawer and slide the tape with labels through the slot under the lamella. Push a partition to the side of the roll and place another roll next to it. Repeat these steps until you fill the dispenser with rolls. Push the drawer back to its position.

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EN

Handling partitions

Removing a partition: Draw out the Anelabel drawer. Release the partition by pulling it to the right. Once released, it is easy to remove it completely from the Anelabel dispenser.

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Label unwinding

Two methods of use:

A If you need to write on a label, tear it off including the supporting tape and only peel it off the tape after you write on it.

B If you do not need to write on a label, peel it off by gently pulling the supporting tape downwards and then stick it on.

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EN

Positioning more than one dispenser in a hall

Dispensers are safe to be positioned on top of each other or next to each other.

Make sure that the higher dispenser is of the same size as the lower one or that the shorter model Anelabel 26 goes on top of the longer Anelabel 36. Never allow a dispenser to “overhang”.

We recommend that only two rows are formed from Anelabel dispensers placed on top of each other.

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EN

Removing the lamella and cleaning the dispenser

The lamella that holds the Anelabel label tapes is fixed with two bolts at its ends. Slacken these screws with the attached wrench and unscrew them with your fingers. We recommend using only Anelabel labels in the Anelabel dispenser.

CZ

Obsah balení

odvíječ Anelabel 36 nebo 26

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klíč

manuál

samolepka pro údržbu

NEČISTIT ALKOHOLEM

SK

Obsah balenia

odvíjač Anelabel 36 alebo 26

variabilné priečky

kľúč

manuál

samolepka pre údržbu

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EN

Package contents

Anelabel dispenser 36 or 26


variable partitions

wrench

manual

sticker for maintenance

DO NOT CLEAN WITH ALCOHOL

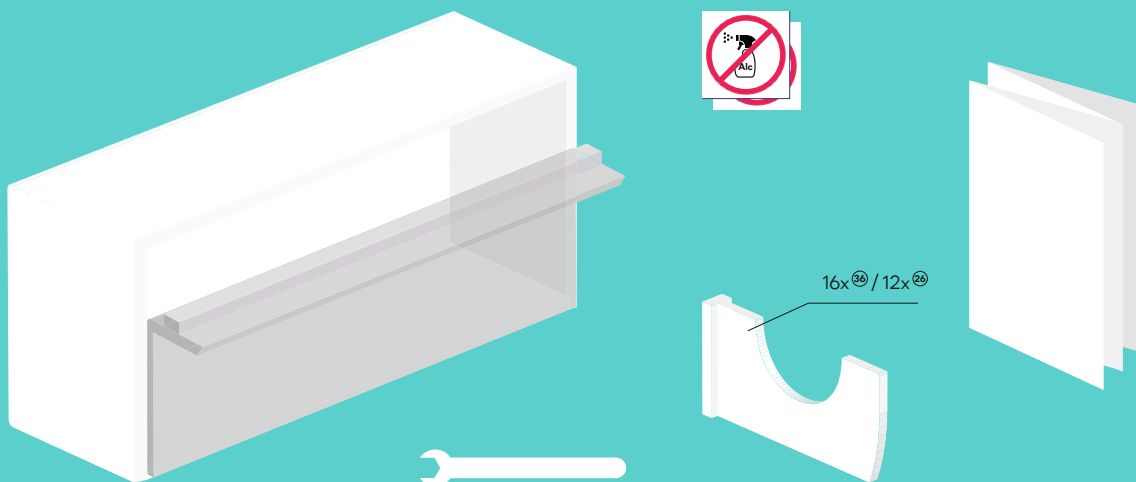③⑥ — Anelabel 36

②⑥ — Anelabel 26

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Maximálny počet roliek v odvíjači

Maximum number of rolls in dispenser

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